



medemagroup



User Manual Mini Crosser M¹ Cabin

Serial number: ____ - _____ - _____ - _____

Delivery date: _____ Year 20_____

This vehicle was supplied by: _____ **Date:** _____ / _____

Contents

Symbols.....	4
Warning!	4
Introduction.....	5
Declaration of conformity	6
Control panel	7
Preparations / Adjustments prior to use	10
Seat rotation.....	12
Steering column	13
Serial number.....	14
Driving the Mini Crosser M ¹ Cabin.....	15
Specific driving situations.....	20
Long hills/difficult ground	22
Anti-tilt wheels / stabilisers	23
General care and maintenance	24
Insurance.....	25
Batteries	26
Charging.....	26
Battery disposal.....	28
Storage	28
Cleaning	28
Fuel for heater.....	29
Heater	29
Windscreen wiper / washer	30
Replacing the wiper blade	31
Replacing the bulb - headlight	31
Changing the wheels.....	33
Fuses	34
Brakes.....	36
Disengagement.....	36
Transporting.....	37
Securing to vehicle floor with belts	38
Transporting by plane	40
Towing	40
Dimensioned drawing, Mini Crosser M ¹ Cabin.....	41
Introduction to seats - Ergo Standard	42
Introduction to seats - Mando	44
Troubleshooting	45
Programming	47
Technical data.....	47
International addresses	50

Symbols



Used in the manual to indicate sections describing situations where extra care is required owing to the risk of personal injury.



Used to indicate sections on electromagnetic compatibility (EMC).

Warning!



For safety reasons the vehicle must not be lent to persons who are not completely familiar with it. The vehicle is designed for one person only.



The Mini Crosser M1 Cabin has been designed for users weighing max. 175 kg. The vehicle can be specially adapted at the factory for users weighing up to 200 kg.

Introduction

Congratulations on your new Mini Crosser M¹ Cabin electric mobility scooter.

You have now taken possession of an electric mobility scooter developed for outdoor driving by active users. It is what is called a Class C vehicle in accordance with the European classification of electric scooters.

For optimum enjoyment of this vehicle - and to avoid breakdowns and accidents - we recommend that you read this User Manual carefully. As a new user, you should pay particular attention to the section entitled "Driving the Mini Crosser M¹ Cabin".

The Mini Crosser M¹ Cabin is designed for safe travel for at least 10 years, up to a max. of 5,000 hours, provided it is serviced and safety-checked every year, corresponding to 500 hours of operation. The service must be carried out by an authorised workshop.



IMPORTANT! For safety reasons it is of the utmost importance that service and safety check intervals are complied with, as this minimises the risk of brake failure and short-circuits in the wiring, which could generate heat and cause a fire.

We offer a wide range of accessories for the Mini Crosser M¹ Cabin that can make everyday life easier for you. You are always welcome to contact us for further information on special accessories and adaptations.

Medema Production A/S is not responsible for any damage or injuries caused by inappropriate or unsafe use of the Mini Crosser M¹ Cabin.

If you have any further questions about the Mini Crosser M¹ Cabin or this User Manual, you are always welcome to get in touch. Our contact details are as follows:

Medema Production A/S

Tel: +45 7010 2054

Email: info@minicrosser.com


Internet: www.minicrosser.dk


NB: Errors and omissions excepted. We reserve the right to update this manual as required.

Control panel

Avoid subjecting the control panel to jarring and knocks. Take care when driving and avoid hitting obstacles.

The manufacturer accepts no liability in respect of the unauthorised opening, adjustment or modification of the control panel.

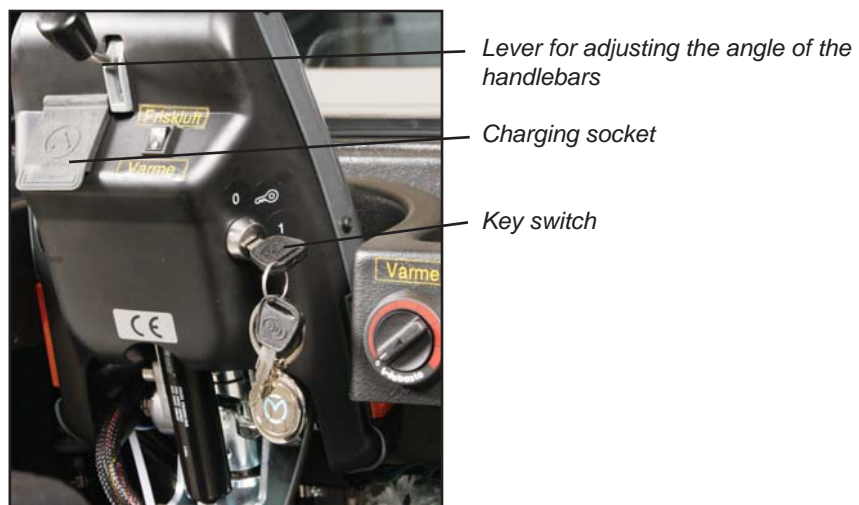

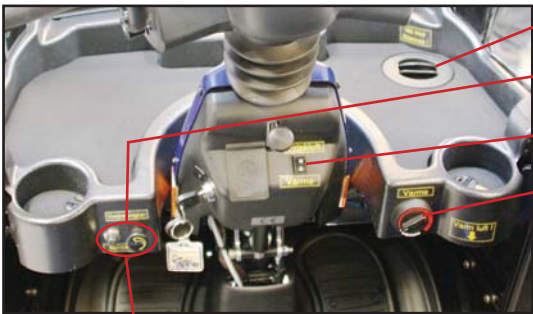

- Air channel
- Windscreen wiper + washer
- Fresh-air/hot-air switch
- Heat regulation



Accelerator
(four-finger control)

Designation	Description
Key switch	To start the vehicle: Turn the key to driving position (1) When the Mini Crosser is turned on, the control system performs a safety check of the electrical system. The check lasts half a second. If the accelerator is operated during this time, the Mini Crosser will not be able to move until you have turned the key to 0 and back to 1.
Speed selector	The tortoise in minimum position indicates the lowest speed range. 0-6 km/h The hare in maximum position indicates the highest speed range. Up to 15 km/h
Battery indicator	Lights up when the key is turned. Indicates, after half a second, the battery charge level. It gives a more precise indication after approx. 1 minute of driving. When the red, amber and green sections are lit, the batteries are fully charged. When the indicator is at the bottom of the amber section, the batteries should be charged as soon as possible. If only the red section is lit or flashing, the batteries must be charged immediately.
Handbrake	Acts on the rear wheel and should only be used as an extra /emergency brake and parking brake. It can be secured in locked position. Also available as a foot brake.
Accelerator	Activate the arm at the front slowly and the Mini Crosser will start to move forwards. The more the arm is pressed, the faster the Mini Crosser will move. When the arm is released, it will flip back to starting position of its own accord and the Mini Crosser will stop. Activating the rear arm will cause the Mini Crosser to reverse. The magnetic brake on the rear wheels will engage when the scooter is stationary and is equivalent to the parking brake on a car. The accelerator can also be used to control speed downhill. In this case the motor will act as a brake.
Light switch	Turns on the front and rear lights. The key switch must be in position 1 (driving position).
Hazard warning lights	Turns on all the indicator lights at once. Also works when the key is removed or turned to position 0 (stop position).
Indicator switch	Left arrow: indicator, left-hand side. Right arrow: indicator, right-hand side. Activate the same switch to turn the indicators off.
Horn	Powerful electric horn. Press the horn switch to activate.
Charging socket	Turn off the key switch during charging. Please note that it is not possible to drive the scooter while the batteries are charging. See also the section entitled "Batteries and charging".
Windscreen wiper	Starts the windscreen wiper. Notice the variable speed adjustment.
windscreen washer	Spraying washer fluid on the front window.
Fresh-air/hot-air switch	Switch to toggle between heated and unheated air.
Heat regulation	Adjustable heat in the cabin. Note: Heat can be turned on even if the key is not in the key switch. Heating ribs should NOT be covered. REMEMBER! turn off the heat when you leave the vehicle. Otherwise the batteries may be flat the next time you want to use it.

Preparations / Adjustments prior to use


Adjusting the height of the seat

Lift the seat off. Pull the release out and adjust the seat tube to the desired position. See the picture on the next page. Check that the seat height is correct and let go of the release. Move the seat tube up/down until the release clicks into a hole.


Make sure you keep your back straight when lifting the seat, which is very heavy. (See below.)




Lift the seat off the seat tube.

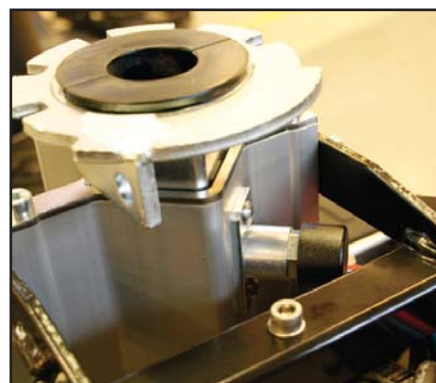

The release button pressed in. Press the button out to release the seat tube.

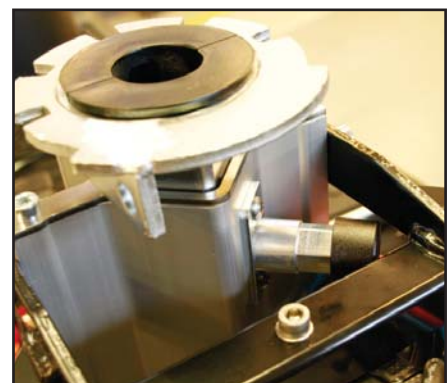

Release button pressed out. Now the seat tube can be moved up/down to the height you wish. Press the release button again and make sure it is fitted in one of the 5 holes in the seat tube.

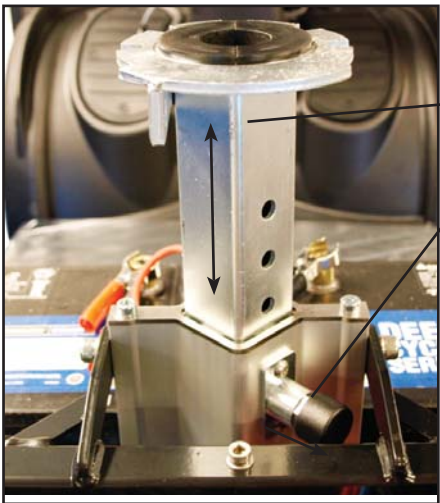

Seat tube

Release

Seat tube in top position. There are five holes for adjusting the seat tube. The release must be in a hole for the seat to be stable.

Seat rotation

It is only possible to rotate the seat in a Mini Crosser with Cabin if both doors are open.

Pull the release lever back. The seat can be rotated 90° to either side. When the lever is released, it engages with the seat and holds it in place at 45° intervals.

Other seats that can be supplied for the Mini Crosser work on similar principles. The release lever is normally mounted on the right, but can be put on the left if so wished.

IMPORTANT!




The Mini Crosser is most stable when the seat is in its lowest position.



Take extra care when lowering the seat on a Mini Crosser using electric seat adjustment. Make sure that nothing is trapped in the space between the seat and the chassis.



1. Release lever for seat rotation.
2. Release lever for seat forward/back.

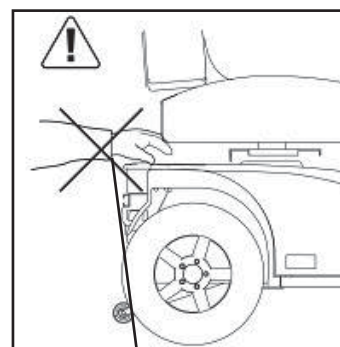

Avoid trapping anything when lowering the seat.

Steering column


Can be adjusted forward/back with the release lever. Pull down on the lever and pull the handlebars closer for a good driving position. Use the lever again and push the steering column forward to make it easier to get out.

The height of the steering column can be adjusted approx. 11 cm up/down. First remove the rubber plug covering the Allen screw. Adjust the height using a 4 mm Allen key.



Adjusting the height of the handlebars.

Lever for adjusting the angle of the handlebars

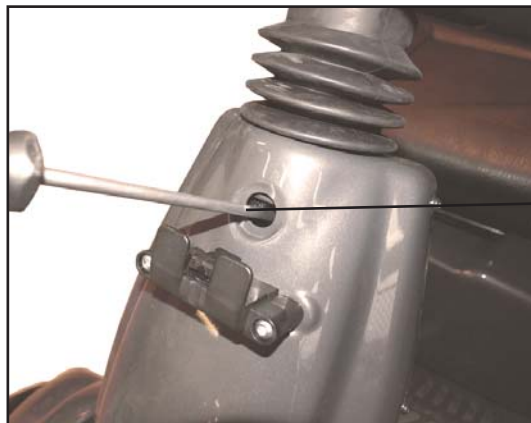


Adjusting the height of the handlebars.

Allen screw for height adjustment

Serial number

All vehicles have a serial number plate showing the year of production, month and serial number. The same serial number can also be found on the front of the User Manual.

Please quote this number when making inquiries about servicing, spare parts, etc.

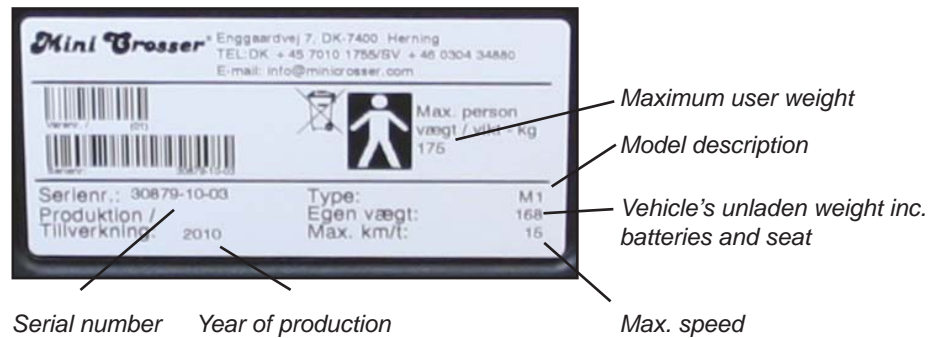

Position of serial number plate on vehicle.

Driving the Mini Crosser M¹ Cabin

Getting in and out

It is important to learn a safe technique for getting in and out of the Mini Crosser.

In general the following method should be followed:

- Make sure that the Mini Crosser M¹ Cabin is off (switch turned to 0) when getting in and out, otherwise the Mini Crosser could start if the accelerator was touched inadvertently.
- Make sure that the brake is on. (Lever for disengaging motor in top position.)
- Put the steering column in vertical position.
- If necessary, raise the armrest.

Note! - Do NOT use the dors as support, when entering or getting out of the vehicle.

For some users the assistance of an attendant can be recommended. The attendant should:

- Take care not to injure him/herself when lifting/lowering/supporting the user.
- Make sure that the Mini Crosser is stable and unable to move. Turn off the Mini Crosser and check that the brake is on.
- Make sure that the seat the user is being moved to is stable.

IMPORTANT!



The Mini Crosser will turn itself off automatically after it has been stationary for 10 minutes even if the key has been turned to driving position.

To restart the vehicle, the key must be turned to stop position (0) and then to driving position (1).



Turn the Mini Crosser off, return the steering column to upright position and raise the armrest.

Braking systems

There are three braking systems on the Mini Crosser M¹ Cabin:

- Motor brake - adjusts vehicle speed also when going downhill.
- Magnetic brake - the magnetic brake is automatic and engages when the Mini Crosser stops. In an emergency, the Mini Crosser can be stopped instantaneously by turning the key switch. Please note that this will cause very sharp braking. The rear wheels will lock.

Must NOT be used in the normal course of driving.

The brake must never be disengaged mechanically using the disengagement lever on a slope. This function is only designed for use when pushing the Mini Crosser on a flat road.

- Electric safety brake - if the brake is somehow disengaged on a slope/hill, the Mini Crosser will brake automatically when it reaches a certain speed. There is an electric safety feature in the control system, which works even if the battery is disconnected. This is also the reason why the Mini Crosser cannot be towed at more than 5 km/h. See the section about towing.
- Handbrake - intended as an emergency brake and parking brake. It must be operated with caution when driving in slippery conditions and downhill.



When using the handbrake as a parking brake, lock it in braking position by pressing the button in while applying the brake.

To release the brake, press the button again.

Handbrake lock

General safety advice

Make sure that the backrest is upright and the seat is as low as possible.

Positioning belts are recommended if the user is unable to maintain a good driving posture independently.

Adjust your driving to road conditions. Take account of light,

traffic and weather. Be particularly careful when driving in the dark or in bad weather, such as rain or snow. Avoid driving on gradients with poor surfaces, such as: snow, ice, new-mown grass, wet grass and wet leaves.



Never drive when under the influence. This applies not only to alcohol, but also to drugs and medicines.

Reduce speed immediately if you feel you are losing control.

Always use the indicators when changing direction.

Check that lights and indicators are working before driving off. Use your lights when driving after lighting-up time.

As far as possible, grip the handlebars firmly with both hands.

Take care not to have anything in the basket at the front that might inadvertently operate the accelerator.



WARNING! Do not lower the handlebars so far that the accelerator can hit you on the leg when you turn the vehicle and so be operated in advertently. If the handlebars need to be very low, a twist grip accelerator is recommended. This is particularly relevant if leg supports are being used.



ALWAYS switch the scooter off when it is not in use. Turn the key to 0.

Driving

Even though the Mini Crosser is very stable, it can tip over. Avoid sudden changes of speed and direction when travelling at high speed, on poor surfaces and, not least, on slopes.

For short distances the Mini Crosser can drive up steeper gradients than it has been tested as dynamically stable for. The same applies to driving down such gradients. In such cases there is an increased risk of the Mini Crosser tilting and even tipping over. So be extra careful in following the driving tips given below.

Anti-tilt wheels are recommended for driving in very hilly terrain. (Optional extra)

New users are urged to practise the following in an area where there is no other traffic:

- Set the Mini Crosser to low speed. Drive forwards and backwards. Gradually turn the speed selector up and feel the change in the speed of the Mini Crosser.
- Practise starting and braking. Get used to the Mini Crosser's response time.
- Practise driving in a narrow space, similar to inside a shop or through a door.
- Practise turning, and get a sense of how much space is required. Always drive slowly when turning. Practise reversing too.
- Practise cornering and driving over obstacles and on slopes. Always drive straight up/down kerbs and ramps. Never at an angle. See the illustrations on the following pages.
- Try braking at different speeds and notice the stopping distances.
- If possible, try driving on different surfaces (road, grass and gravel).
- Practise assessing how far you can drive on a single battery charge. Note how quickly the battery indicator changes from green to amber to red.



Note!

The driving distance of the scooter will be reduced when driving in hilly districts, into a head wind, in cold weather and with low tyre pressure.

Traffic regulations

The traffic legislation for scooters varies from country to country. Before starting to use the vehicle outdoors, it is the user's responsibility to familiarise him/herself with the relevant legislation.

The following rules apply in Denmark:

Mobility vehicles such as the Mini Crosser with a maximum speed of up to 15 km/h count as cycles in traffic legislation and the traffic regulations for cycles must be followed at speeds in excess of 6 km/h.

At speeds of up to 6 km/h the Mini Crosser is regarded as a "pedestrian", and it is permissible to travel on the pavement at

this speed. "Tortoise" speed corresponds to approx. 6 km/h. Lights must only be used during lighting-up time. Crash helmet and seat belts are not compulsory.

Electromagnetic compatibility



If the Mini Crosser starts making involuntary movements or if the brakes are released, turn the Mini Crosser off as soon as it is safe to do so. In certain circumstances a Mini Crosser can set off shop alarms.

The Mini Crosser satisfies the requirements for the use of scooters in an environment with electromagnetic noise. There may, however, be rare situations in which electromagnetic noise can affect the Mini Crosser. Sources of such noise include radio and television stations, amateur radio transmitters and mobile phones.

If such equipment is being used close by, it is recommended that the Mini Crosser is switched off. Mobile phones should not be used while driving.

When driving in traffic...

Be particularly aware of the following when driving in traffic:

- The Mini Crosser is a low vehicle and not always easy for other road users to see. Make quite sure that other road users have seen you before driving onto the highway.
- Be aware of the blind spots and look around them before changing direction.
- Keep an eye on traffic behind you. Keep well over to the side of the road when driving on busy roads.
- Turning right and left at crossroads. Be aware of cyclists and pedestrians. Follow the rules of the road for cyclists.
- How quickly things are happening. How long do the lights stay green? How quickly are cars approaching? etc.

Specific driving situations


Up kerbs

- Stop at right angles to the kerbstone about 5-10 cm away from it. Keep an eye on other road users.
- Lean forwards.
- Accelerate moderately so that your vehicle can overcome the obstacle. Do not stop halfway, but reduce speed once the vehicle is up.
- If the kerb is too high, do not try again, but find an alternative route.





Down kerbs

- Lean back.
- If you are driving onto a road with traffic, keep an eye on other road users.
- Drive forwards and down the kerb at low speed. Make sure that your anti-tilt wheels (if fitted) do not catch on the edge.

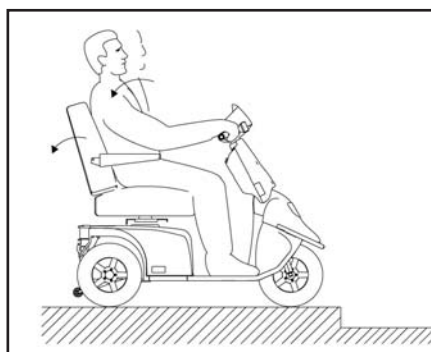



Up a ramp/hill

- There is a risk of tipping over backwards if the seat is pushed back when you start driving up a slope, e.g. a ramp.
- Pull the seat forward! Check that any ramp is stable.
- Lean forwards.
- Accelerate moderately so that your vehicle can overcome the obstacle. Do not stop halfway. Reduce speed once the vehicle is up. If you need to start on a hill, accelerate slowly so as not to tip over backwards.

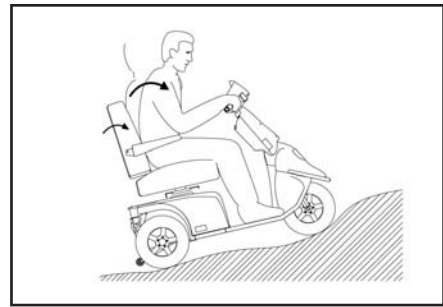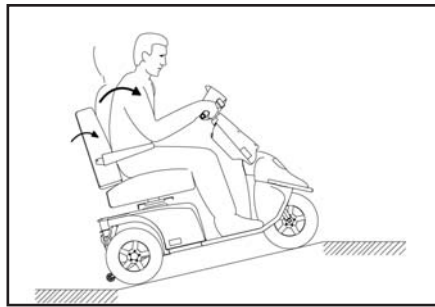

Down a ramp/hill

- Check that any ramp is stable.
- Lean back.
- Drive slowly down. Avoid stopping midway on short, steep slopes. On long hills, it is advisable to stop every now and again if you feel your speed is getting too high.

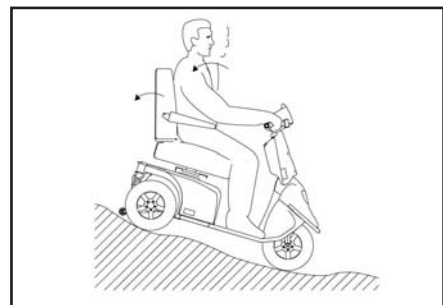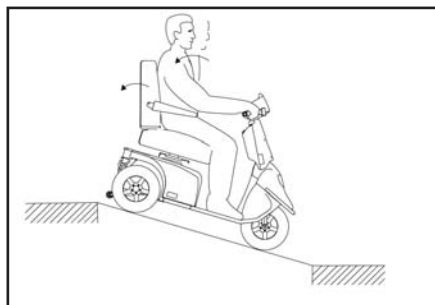

Across a slope

- Lean into the slope.
- Avoid sudden and sharp movements. This is particularly relevant when reversing.
- Always drive at low speed.

Long hills/difficult ground

The Mini Crosser M¹ Cabin is at risk of overheating on very long and steep hills or on very soft ground, possibly also combined with high user weight.

To protect the Mini Crosser motor from overheating, there is a built-in thermoswitch, which initially sets the Mini Crosser to half speed.

However, if you carry on moving over difficult ground, the Mini Crosser will at some stage stop completely, and the battery indicator will flash with 6 lamps. The Mini Crosser must be left to cool down for 3-5 minutes before it can start again.

To reset the fault, switch off the Mini Crosser then turn it on again.

If the Mini Crosser has not had long enough to cool down when it is restarted, it will initially only run at half speed.

Note:

The situation described above will never happen in normal use. It only happens if the conditions are extremely challenging.

Anti-tilt wheels / stabilisers

The Mini Crosser M¹ Cabin is a very stable vehicle. HOWEVER, in the case of incorrect weight distribution or careless driving there is a risk of tipping over.

We therefore recommend fitting anti-tilt wheels in such circumstances. (See picture below)

(Anti-tilt wheels / stabilisers are optional extras.)



Anti-tilt wheels

General care and maintenance

A Mini Crosser M¹ Cabin does not require much maintenance. It should be kept in a generally good condition, however. The following should be checked regularly:

- Tyre pressure (if pneumatic tyres are fitted)
- Tyre wear
- Keep the control panel, the charging socket and the electronics box under the seat dry.
- Battery charging



Never wash the Mini Crosser with a high-pressure cleaner or direct water jet! This could damage the Mini Crosser's electronics.

To keep the Mini Crosser in good condition safety-wise, we recommend the following regular checks:

Daily:

- Test the indicators and driving lights before using the Mini Crosser in the dark or poor visibility.

Every three months:

Test the brakes and motor disengagement

With the disengagement lever up, it must not be possible to push the Mini Crosser.

Test the brake disengagement function

When the brake disengagement lever is down, the battery indicator should flash to show an error if the Mini Crosser is turned on. In this case the Mini Crosser must not be able to move when the accelerator is activated.

Test the handbrake.

Apply the handbrake for a couple of seconds at low speed. This will ensure that the lever arm and brake shoes do not seize up.

Lubricate the lever arm on the brake hub with acid-free oil - left-hand rear wheel.

Test the windscreen wiper.

Spray washer fluid on the window and check that the wiper blade works properly.

Annual

The Mini Crosser M¹ Cabin is designed for safe travel for at least 10 years, up to a max. of 5,000 hours, provided it is serviced and safety-checked every year, corresponding to 500 hours of operation. The service must be carried out by an authorised workshop.



IMPORTANT! For safety reasons it is of the utmost importance that the servicing and safety check intervals are complied with, as this minimises the risk of brake failure and short-circuits in the wiring, which could generate heat and cause a fire.

(For further information see the Service Manual.)

Insurance

In the eyes of the law a Mini Crosser M¹ Cabin with a maximum speed of 15 km/h is a cycle, so separate insurance is not required.

Most contents/home insurance policies include third-party liability insurance for cyclists and so also cover Mini Crosser M¹ Cabin users.

We recommend that you talk to your insurance company about this when the vehicle is delivered. If necessary, comprehensive insurance will have to be taken out separately.

Batteries

The Mini Crosser uses sealed, maintenance-free GEL batteries (Exide 12 V/56 Ah or AGM 12 V/75 Ah).

They do not generate gas and do not have to be topped up with water.

Charging



Please NOTE that the Mini Crosser can be equipped with several types of charger (ask your dealer for information on the various types).



IMPORTANT! Only ever use a charger designed for charging dry maintenance-free batteries. The max. charging current is 12 A.

If charging is to take place outdoors, an enclosed charger without a fan should be chosen.

The battery manufacturer recommends charging the batteries at temperatures between +10° and +30°C in order to achieve the charging times specified in “Technical data”.

Charging will take longer at temperatures below +20°C. This is because the battery finds it more difficult chemically to absorb the current.



Please NOTE that the capacity of the batteries will reduce over time and at low temperatures. In practice this means that vehicles with old batteries have a shorter driving distance than they had when the batteries were new.


Battery capacity at -10°C is about half the equivalent capacity at +20°C.

Avoid draining the batteries completely. We recommend charging the batteries in a heated room. If it is not possible to do this every day, it is a good idea to charge the Mini Crosser in a heated room (approx. 20°C) once a week.

New batteries do not reach full capacity until they have been charged and discharged about 20 times.



It is **IMPORTANT** to fit the batteries correctly. The battery terminals and snap locks are marked +/- . They must be fitted as shown in the sketch below. Make sure that the snap locks are properly closed. For the same reason there must not be any burrs on the terminals.

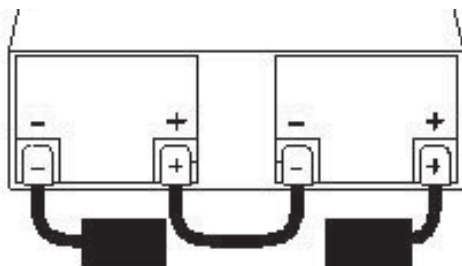

The Mini Crosser should be charged while not in use. The charger supplied from the factory switches over to trickle charging (very low power consumption) automatically when the batteries are fully charged. You can therefore leave the charger connected until the Mini Crosser is next used. The charger cannot overcharge the batteries.


The charger will flash until charging is complete. Then it will show a steady light.



The charger must not be placed on the seat during charging.



IMPORTANT! Never use charging devices other than those supplied from the factory without first contacting the dealer.

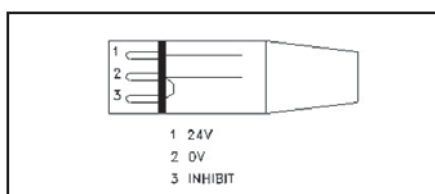

NEUTRIK NC3MX charging plug



Connect the charging plug to the 3-pin socket on the steering column. The charging socket is located under the protective cover.

Battery disposal

Used batteries must be disposed of through your supplier or at a recycling centre.

Take care when handling leaky batteries, as the gel contains corrosive acid.

INFO!

New batteries can be purchased from Medema Danmark A/S.

Storage

The scooter should be stored and charged under cover, preferably at temperatures above 0°C.

Note!



The charger must be kept dry, but should not be covered when in use.

In the case of long-term storage we recommend covering the Mini Crosser M¹ Cabin to protect it from dust, rain and sunlight.

Cleaning

Clean the Mini Crosser M¹ Cabin with ordinary autoshampoo. If necessary, wipe dry with a chamois leather.

Wipe the covers dry and polish with car wax.

Rinse with a garden hose with a soft jet.



IMPORTANT! Never use a high-pressure cleaner or hose, as it could damage the Mini Crosser's electronics.

Fuel for heater

The tank for fuel is placed behind the seat, and filled from the outside.

Use **ONLY** paraffin or diesel.




In the front, right side of the Mini Crosser M1 Cabin, you find the the exhaustion from the heater.



NOTE! It gets **VERY** hot.

Exhaustion from heater.

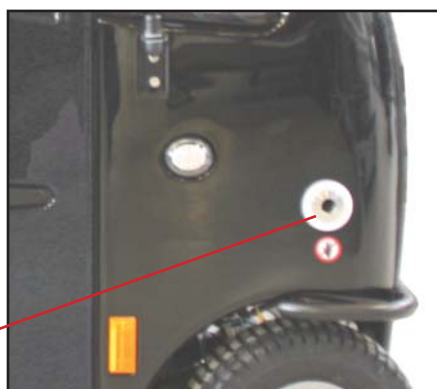

Heater



Knob to:
- turn heater on/off
- adjusting the temperature
- remove fault code.

Indicator light for on/off and fault code readout.

For more information: see Webasto user guide.



IMPORTANT!

The heating ribs by the window and by the legs should NOT be covered up when the heating is on. That will cause overheating of the heater



Windscreen wiper / washer

Start the windscreen wiper by turning this knob.

Notice the smooth speed adjusting.

Windscreen wiper - turn the knob.

Washer - press this button



The Container for washer fluid is placed under the front cover.

The container for the washer fluid.




Replacing the wiper blade



To exchange the wiper blade, do the following:

Screw this screw out.

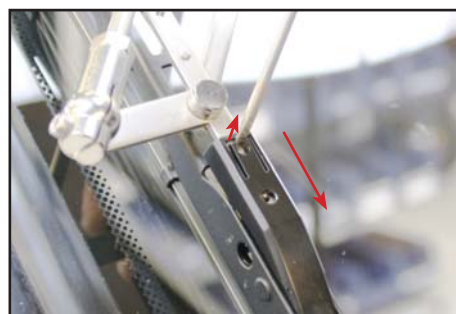


Lift carefully, so you can pull the wiper blade over the pin.

Mount a new in reverse order.

Replacing the bulb - headlight



Open the front cover, by turning the handle 90° counter clockwise, and pull.

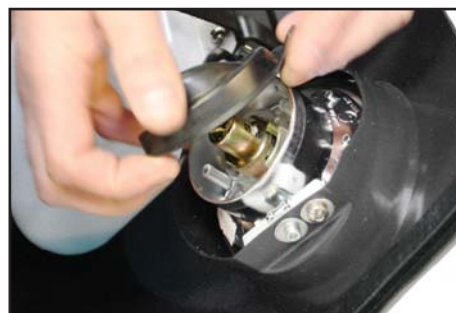

Remove the protective cap.



Press down and turn counter clockwise.-



and lift out.

Now you have access to the bulb.



take the bulb out.

Mount a new in reverse order.



NOTE!

Be aware that drain list are not trapped when the front cover is closed.

It **MUST** cover the edge of the front cover.

Changing the wheels

If you get a puncture in one of the pneumatic tyres or if a tyre is so badly worn that it needs to be replaced, follow the instructions below.

Tyres and inner tubes can be purchased from the authorised dealer who supplied the Mini Crosser.

The Mini Crosser must be TURNED OFF before you start.



Changing the wheels

5 mm Allen key

- Undo the five bolts.
- Take the wheel off.
- When the wheel is put back on, the spring washers must be refitted between the wheel rim and bolts.

The bolts must be done up tightly.

REMEMBER!

- to let all the air out before taking the wheel apart to repair a puncture
- to secure the bolts with spring washers

Fuses

The Mini Crosser M¹ Cabin has the following protection systems:


- Automatic overload protection, which limits the current to the motor. This protection is integral to the control system and cannot be adjusted. It prevents both the control system and the motor from being overloaded. In the event of overload the current to the motor is reduced - full speed and traction can be resumed after the motor has cooled down for 2-5 minutes.
- The control fuse protects against mains faults. It is located under the rear cover.
- The main fuse protects against short-circuits in the main current circuit. The fuses are located under the rear cover behind the batteries. Fuse type DIN 2581.
- If the ignition key is turned at the same time as the accelerator is operated, the vehicle will not move off for safety reasons.
- The lights and horn are protected by electronic fuses, which protect against any short-circuiting that may occur. To prevent accidental short-circuiting when a bulb is changed, the Mini Crosser must be turned off when the new bulb is fitted and then turned on again afterwards.



Take the rear cover off. The plastic cover can be removed by squeezing the sides. Now the fuses can be replaced.



Open the front cover, by turning the handle 90° counter clockwise, and pull.

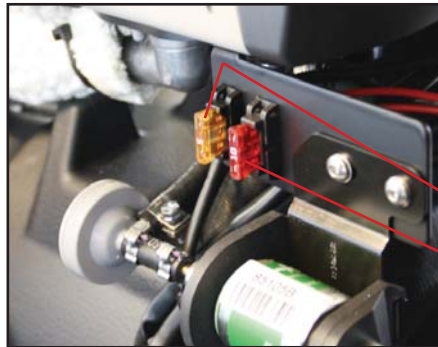

Under the front cover you will find the fuse for the heater and windscreen wiper.

Fuse for windscreen wiper.

Fuse for heater.



NOTE!

Be aware that drain list are not trapped when the front cover is closed.

It **MUST** cover the edge of the front cover.

Brakes

There are three braking systems on the Mini Crosser M¹ Cabin:

- Motor brake
 - Magnetic brake
 - Handbrake
-
- Motor brake: adjusts vehicle speed also when going downhill.
 - Magnetic brake - the magnetic brake is automatic and engages when the Mini Crosser stops. In an emergency, the Mini Crosser can be stopped instantaneously by turning the key switch. Please note that this will cause very sharp braking. The rear wheels will lock.
Must NOT be used in the normal course of driving.
The brake must never be disengaged mechanically using the disengagement lever on a slope. This function is only designed for use when pushing the Mini Crosser on a flat road.
 - Electric safety brake - if the brake is somehow disengaged on a slope/hill, the Mini Crosser will brake automatically when it reaches a certain speed. There is an electric safety feature in the control system, which works even if the battery is disconnected. This is also the reason why the Mini Crosser cannot be towed at more than 5 km/h. See the section about towing.
 - Handbrake: intended as an emergency brake and parking brake. It must be operated with caution when driving in slippery conditions and downhill.

Disengagement


- Turn the key to (0) position.
- Push the disengagement lever at the back down.
The motor brake is now disengaged and the vehicle can be pushed or towed, but not driven.

NOTE!



The motor brake must never be disengaged on sloping terrain.

Once the motor brake has been disengaged, the Mini Crosser can only be braked with the hand brake.

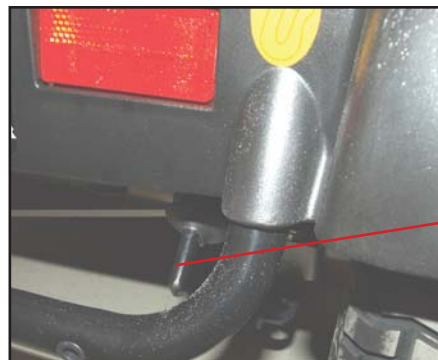

Push the disengagement lever down to disengage (only handbrake works) and push up to reactivate the motor (normal driving now possible).

Disengagement lever

Transporting

The Mini Crosser m¹ Cabin must always be restrained and the handbrake locked during transport.

Avoid lifting by the covers.

If you just need to lift the scooter slightly, take hold of the rear bumper and the front bumper.

Secure it in the vehicle with belts attached to the two eyes at the front and two at the back. All the eyes are marked in yellow. See the section entitled “Securing to vehicle floor with belts”.


Securing to vehicle floor with belts


Dahl Engineering belt set for securing in motor vehicles.
Item No. C2-0242

ALWAYS use four belts at the back and two at the front.

The belts must always be attached to approved fittings in the vehicle and the four eyes welded to the scooter.

The belts **MUST** be attached within the angles shown in the picture for optimum security.

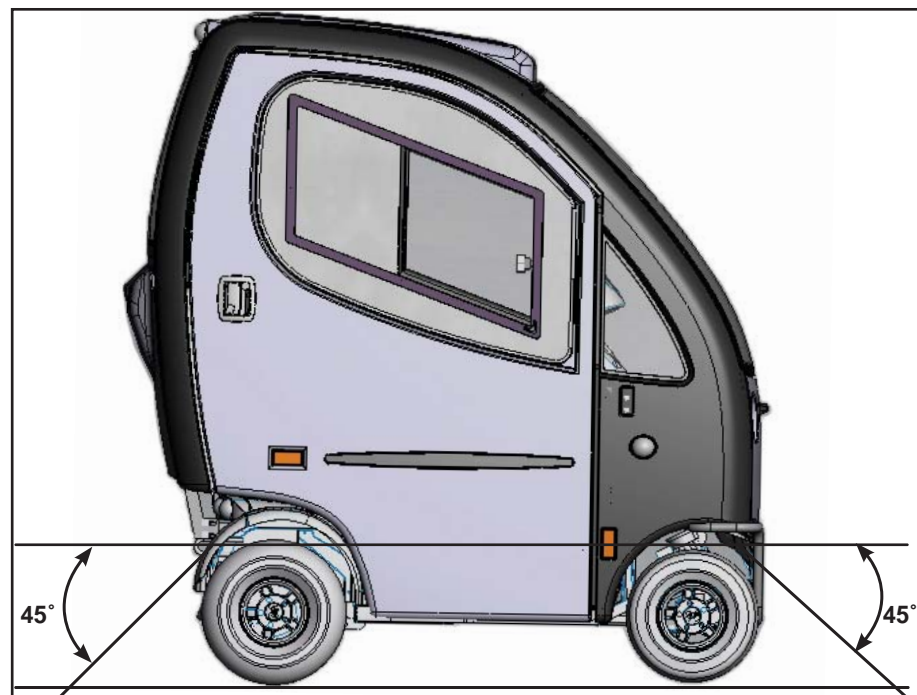

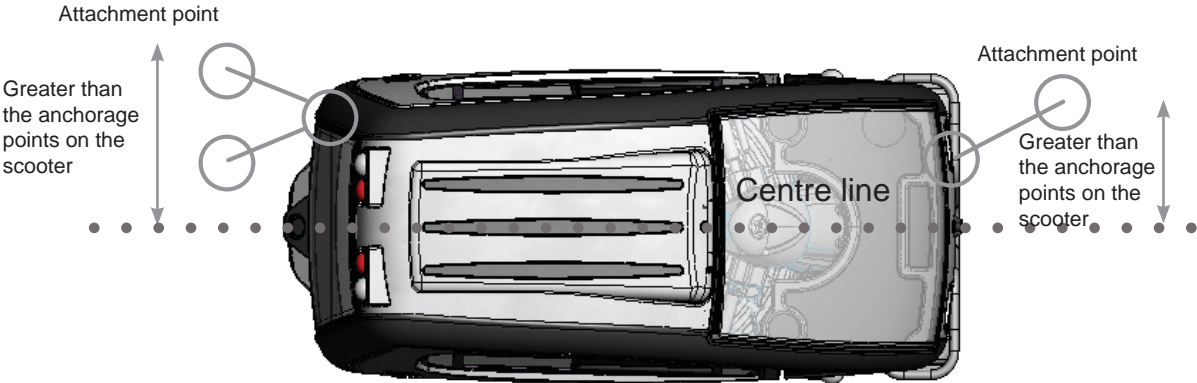

Transporting by plane

If the Mini Crosser is to be transported by plane, the airlines require:

- the batteries to be flight-approved
- the air to be let out of the tyres
- the battery leads to be disconnected (not always, but frequently)

The seat and cover have to be removed to disconnect the battery leads.

A battery declaration for air travel can be found on the Mini Crosser website:

http://www.minicrosser.dk/Download_brochurer.asp

Towing



If you should be unfortunate enough to break down, the Mini Crosser can be towed or pushed. The Mini Crosser must always be turned off and the motor brake disengaged during towing. See the section on Brakes.



If the Mini Crosser is to be towed, secure a rope to the tow fitting on the front - marked with a yellow “hook mark”. Do not tow faster than 5 km/h. The scooter will generate electricity when it is towed, with the motor acting as a dynamo. If it is towed at more than 5 km/h, there is a risk of the motor generating enough electricity to damage the scooter and, in the worst case, cause a fire.

The Mini Crosser M¹ Cabin will attempt to brake, if it is towed faster than 5 km/h.

Dimensioned drawing, Mini Crosser M¹ Cabin

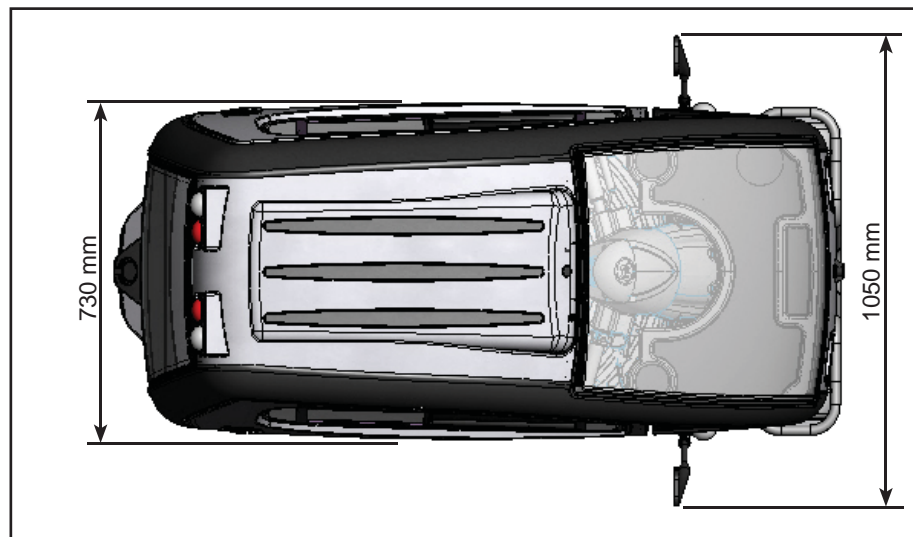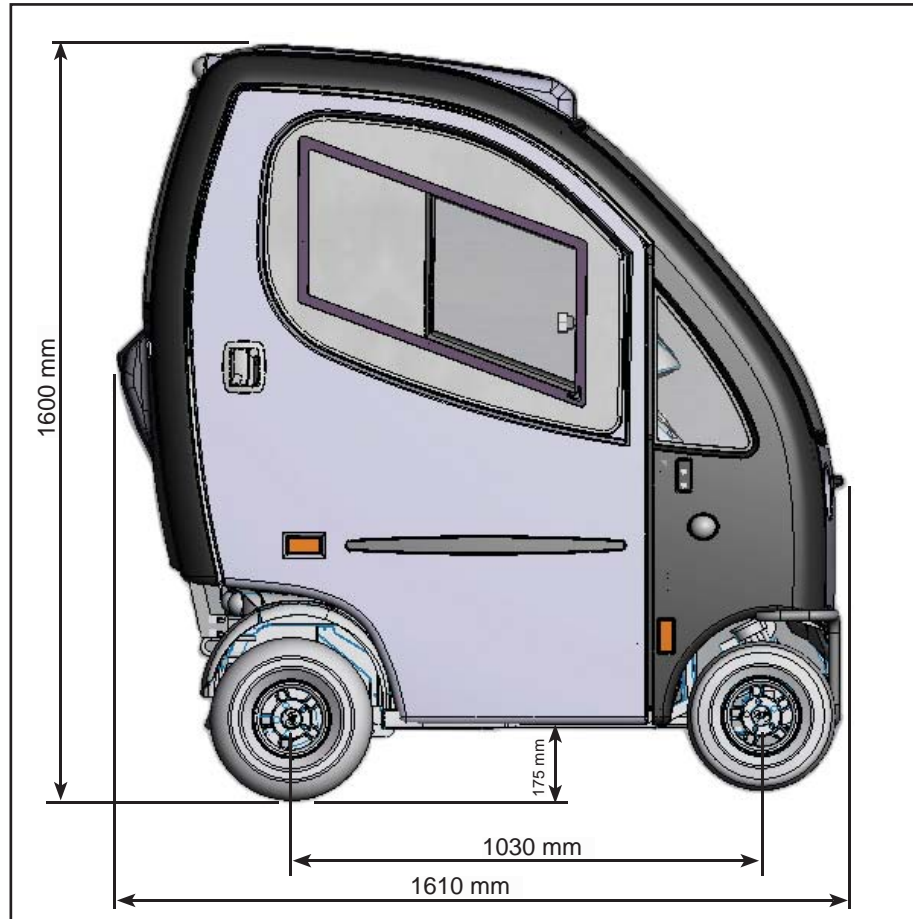

Introduction to seats - Ergo Standard



Lever for seat rotation

Pull the lever back to release the seat. The seat can then be turned 90° to either side with a stop every 45°.

The lever is spring-loaded and the seat locks automatically when the lever is released.

This lever is located on the right as standard, but can also be put on the left.



Lever for seat forward/back

Pull the lever out/up to release the seat on the slide rail. The seat can then be moved forward or back as required.

When the lever is released, the seat automatically locks at the nearest stop.

There is a chock block on the front and back of the slide rail.



Adjusting the angle of the armrest

The adjusting screw makes it possible to alter the angle of the armrest by about 15°.


The arm can be folded right up to make it easier to get in and out.

(Height adjustment of the armrest is an optional extra.)




Adjusting the angle of the backrest

The angle of the backrest can be adjusted by about 30°. This can be done by undoing the adjusting screw and changing the position of the spacer. Do the screw up again after making the adjustment.

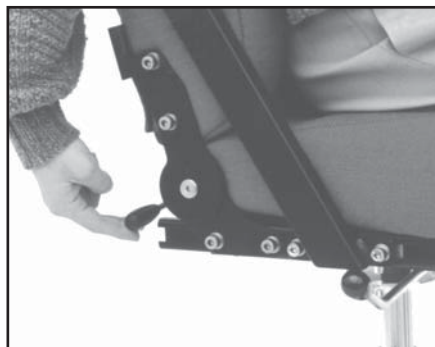


Optional extra:

The seat can be fitted with a gas cylinder for adjusting the angle of the backrest.

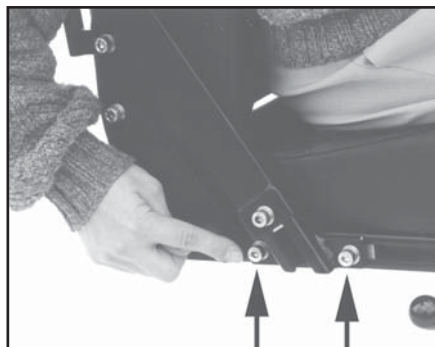

Armrest forward / back:

The armrest can be moved forward / back on the C-rail. This can be done by undoing the screws indicated. Do the screws up again after making the adjustment.

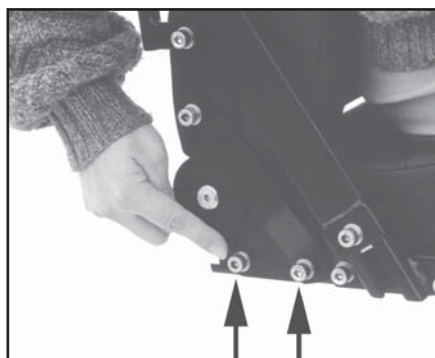

Backrest forward / back:

The depth of the seat can be adjusted. This can be done by moving the backrest forward and back while the seat cushion is held in position. Undo the screws indicated on both sides. The backrest can

then be moved as far back as the edge of the rail and about 15 cm forward. Do the screws up after making the adjustment. The screws must be at least 2 cm from the end of the C-rail.

Introduction to seats - Mando



Adjusting the angle of the backrest:

The angle of the backrest can be adjusted by about 30° by raising this lever.




Adjusting the angle of the armrest:

The angle of the armrest can be adjusted by turning this screw.



Adjusting the height of the armrest:

The armrest can be raised/ lowered by undoing the two screws and moving the bracket one hole up or down.

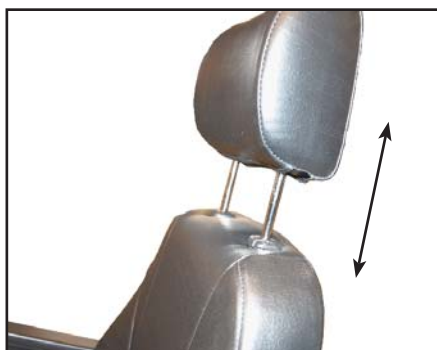

Headrest:

The headrest can be raised and lowered steplessly.

Troubleshooting

The following is a list of various problems that, in our experience, may occur. The list gives possible causes and remedies.

Problem	Possible causes	Remedy
The Mini Crosser will not go. The battery indicator is not lit.	The key has not been turned to start. The batteries are completely flat. The control fuse has blown. The main fuses have blown.	Turn the key and wait 5 sec. before activating the accelerator. Charge the batteries. Change the fuse. Contact supplier.
The Mini Crosser will not go, but the battery indicator is lit.	The Mini Crosser has been overloaded. The handbrake is on. There is a fault in the electronics. The batteries are flat. The charging plug has not been removed.	Wait approx. 1 min. before trying again. The vehicle must be turned off (see section on Fuses). Release the handbrake. Contact supplier. Contact supplier. Remove the charging plug.
The driving speed is too low.	The speed selector is on slow. The electronics are overloaded. There is too little air in the tyres.	Change to a faster speed. Stop and wait a few seconds before starting. Pump the tyres up to the right pressure.
The driving distance per charge is too short.	There is a problem with the batteries. Low temperature. There is a problem with the charger. There is too little air in the tyres. The charging method is wrong.	Charge the batteries and check that the green lamp on the charger lights up before driving off. Contact supplier. Pump the tyres up to the right pressure. Read the section on Charging in the User Manual.
The charging lamp on the charger does not light up when the charger is connected to the mains and the Mini Crosser.	No power to the switch. Fault in cable. Fault in charger.	Turn the switch on. Contact supplier. Read the operating instructions for the charger. Contact supplier.
The "ready" lamp on the charger does not light up even though the charger has been on for 10-12 hours.	There has been a power cut. The charger is doing a top-up charge. There is a problem with the batteries. There is a fault in the charging plug for the Mini Crosser M ¹ .	Reconnect the charger and repeat the charging process. Check again half an hour later. Contact supplier. Push the charging plug all the way in and repeat the charging process. Read the operating instructions for the charger.

Problem	Possible causes	Remedy
The "ready" lamp on the charger lights up even when partly discharged batteries are connected.	The fuse in the charger has blown. The switch in the charging plug is malfunctioning.	Contact supplier. Contact suppliers - read the operating instructions for the charger.
The charger lamp is showing an error.	The charging plug has not been inserted or there is a mains fault. The battery voltage is too low for charging to start.	Push the charger plug in or contact the supplier. Read the operating instructions for the charger - or contact the supplier.
The heater doesn't work.	Need paraffin/diesel. The fuse has blown	Put on paraffin og diesel. Change the fuse. Read the Webasto user guide for more information.
The washer doesn't work.	Washer fluid i missing. The fuse has blown.	Put on washer fluid. Change the fuse.
The windscreen wiper doesn't work.	The fuse has blown.	Change the fuse.

If there is an electronic fault, a number of lamps in the battery indicator will light up. The following table shows what they mean.

Lamps	Causes	Remedy
1 lamp	The batteries need charging or the connection to a battery is poor.	Check all connections between the controller and batteries. If these are fine, try charging the batteries
2 lamps	Poor connection to the motor.	Check the connection between the motor and controller. Contact supplier.
3 lamps	Short-circuit between the motor and a battery connection.	Contact supplier - say how many lamps are lit.
4 lamps	Not in use	
5 lamps	Not in use	
6 lamps	The S180 is prevented from driving. Inhibit 2 is active. This may be due to the battery charger being connected, the plug to the thermal switches being disconnected, a fault in the thermal switch or the scooter is overheating.	Stop the unit and wait 3-5 minutes before restarting. Turn the key to 0 and then to 1 to reset the fault. Contact supplier.
7 lamps	Accelerator fault.	Make sure that the accelerator is in neutral when the Mini Crosser is turned on.
8 lamps	Controller fault.	Contact supplier.

Lamps	Causes	Remedy
9 lamps	Poor connection to motor brake. Magnetic brake is disengaged.	Check the connections from brake to controller, or push the brake lever up. Check that the vehicle is not disengaged.
10 lamps	The controller has been supplied with a high voltage in excess of 38 V. Usually seen in the case of a poor connection.	Check all connections from the batteries to the controller.
Running lights	Charger is connected.	Remove charger.

Programming



NOTE! For safety reasons, modifications may only be made by trained personnel, such as Mini Crosser service engineers and consultants or authorised service personnel at mobility centres, etc.

Technical data

Technical data	M ¹ Cabin
General information:	
Transport height without seat (Steering column folded down)	160 cm
Total length: Total length incl. anti-tilt wheel: Total width:	161 cm 169,5 cm 73 cm
Total weight incl. batteries and Ergo Std seat.	243 kg
Weight without seat (Ergo Standard 45 cm).	222 kg
Static stability in all directions	15° - 26%
Dynamic stability in all directions at minimum speed	10° - 17%
Climbing ability with person weighing 175 kg	15° - 26%
Kerb climbing	12 cm
Max. speed	15 km/h
Braking distance 10 km/h 15 km/h	2.0 metres 3.5 metres
Turning radius	184 cm
Max user weight - standard	175 kg

Technical data	M¹ Cabin
Max user weight inc. accessories HD version.	200 kg
Tyre dimensions (6 ply)	3.00-6 max. outside diameter 340 mm
Tyre pressure	2.8 bar
Classification, ISO	Class C
Seats:	
Effective seat width: Ergo Standard without armrests Ergo Standard with armrests Spinalus Mando Ergo child / junior	40,45,50 cm 55 cm 40, 45, 50 cm 48 cm 35, 37 cm
Effective seat depth: Ergo, Spinalus and HD seats Mando Ergo child / junior	32-52 cm 48 cm 20-30 and 25-40 cm
Seat height: Ergo standard / HD Spinalus Mando Ergo child / junior	48 cm 56 cm 53 cm 36-41 cm
Height: front edge of seat to ground Ergo Standard with slide rail Ergo Standard without slide rail Mando seat with slide rail Mando seat without slide rail	64 - 74 cm 61 - 71 cm 64 - 74 cm 61 - 61 cm
Height: footplate to front edge of seat Ergo Standard with slide rail Ergo Standard without slide rail Mando seat with slide rail Mando seat without slide rail	45 - 55 cm 41 - 51 cm 45 - 55 cm 41 - 51 cm
Angle of backrest Ergo Standard / HD (manuel) Ergo Standard / HD (EL) Spinalus Mando Ergo child / junior	Ca. -5° to +20° Ca. -5° to +30° Ca. -5° to +20° Ca.+5° to +30° Ca. -5° to +20°
Batteries:	
56 Ah batteries	Standard
Maximum driving distance with new batteries at +20° on flat, firm surface	35 km
75/80 Ah batteries	Option
Maximum driving distance with new batteries at +20° on flat, firm surface	50 km

Technical data	M ¹ Cabin
110/115 Ah batteries	Option
Maximum driving distance with new batteries at +20° on flat, firm surface	60 km
Optimum battery capacity is reached after approx. 20 charges / discharges.	
Battery type: Standard: Option: Option:	2 x 12 V / 56 Ah 2 x 12 V / 75 Ah 2 x 12 V / 110 Ah
Max. battery dimensions in cm.	26.5 x 17 x 22 28.4 x 26.7 x 23
Battery weight, 2 batteries 56 Ah 75/80 Ah 110/115 Ah	43 kg 50 kg 80 kg
Energy consumption in kWh, when charging from "empty" 56 Ah 75/80 Ah 110/115 Ah	Approx. 1.5 Approx. 1.5 Approx. 3.0
Charging device, 24 V DC 56 Ah 75 Ah 110 Ah	6 - 10 A 6 - 10 A 10 - 12 A
Approx. charging time at 20° C	8 hours
Lighting:	
Bulb, headlight	24 V - 18 W
Diode, rear light	24 V - 2 W
Diode, indicator	24 V - 2 W
Standard colour	Grey

The Mini Crosser has been tested for the requirements described in EN 12184, which are the basis for CE marking. That means that it has also been tested for the durability requirements (ISO 7176-8) and the fire safety requirements (ISO 7176-16).

International addresses

Australia / New Zealand

Pride Mobility Products
21 Healey Road
Dandenong 3175 Victoria
Australia
Tel: +61 3 9706-4611
Fax: +61 3 9706-4622
Email: prideaustralia@pride-mobility.com.au
Internet: www.pridemobility.com/international/Australia/australia.asp

Austria

Sanitätshaus Gebhardt
Nikolaistrasse 10
D-83022 Rosenheim
Tel: (+49) 08031 15466
Fax: (+49) 08031 34762
Internet: www.sanitaetshaus-gebhardt.de/

Belgien

Mobile Belgium
Blankenbergsesteenweg 14
B-8000 Brugge
Tel: +32 (0)50 317919
Fax: +32 (0)50 311026
Email: info@mobilescooter.be
Internet: www.mobilescooter.be

Denmark

Head Office/Jutland

Medema Danmark A/S
Enggårdvej 7
Snejbjerg
DK-7400 Herning
Tel: +45 7010 1755
Fax: +45 9718 8835
Email: info@medema.com
Website: www.medema.dk

Denmark

Zealand

Medema Danmark A/S
Ådalen 9
DK-4600 Køge
Tel: +45 7010 1755
Fax: +45 5664 3388
Email: info@medema.com

Denmark

Funen

Medema Danmark A/S
Sivlandvænget 6
DK-5260 Odense S
Tel: +45 7010 1755
Email: info@medema.com

Finland

Respecta OY
Tenholantie 12
00280 Helsinki
Myyntipalvelu Tel: +358 (0)207649748
Email : myynti@respecta.fi
Fax :+358 (0)207649735
Huolto - ja varaosapalvelu Tel: +358 (0)207649747
Email : huolto@respecta.fi

France

ICARE
4 rue d'amsterdam
F-91250 TIGERY
Tel: +33 (0)1 60 78 94 84
Fax:+33 (0)1 69 13 09 78
Email: icarejmb@wanadoo.fr
Internet: http://www.icare-jmb.com

Germany

Baden Württemberg

Draisin GmbH
Von-Drais-Straße 35
77855 Achern
Tel (+49) 0 78 41 66 77 0
Fax (+49) 0 78 41 66 77 88
Email: info@draisin.com
Internet: www.draisin.com/

Germany

Baden Württemberg

Elofahrl Palm
Finkenweg 27
D-76571 Gaggenau
Tel: +49 07225 918 831
Fax: +49 07225 988 070
Mobile: +49 0171 314 7726
Email: uwepalm@web.de

Germany

Berlin, Mecklenburg-West Pomerania, Saxony-Anhalt, Thuringia, Brandenburg and Saxony

Conze Elektro-Mobile
Britzer Damm 12
D-12347 Berlin
Tel: +49 030 6789 7969
Fax: +49 030 6789 7968
Email: info@elektro-mobile.de
Internet: www.elektro-mobile.de

Germany

Bavaria

Sanitätshaus Gebhardt
Nikolaistrasse 10
D-83022 Rosenheim
Tel: (+49) 08031 15466
Fax: (+49) 08031 34762
Email: info@ot-gebhardt.de
Internet: www.sanitaetshaus-gebhardt.de/

Germany

Hesse, Rhineland-Palatinate, Saarland and Lower Franconia

Beromobil GmbH
Elektromobile Treppenlifte
Draiser Weg 16
D-65346 Eltville-Erbach
Tel: +49 06123 - 60 57 22
Fax: +49 06123 - 60 57 23
Email: info@beromobil.de
Internet: www.beromobil.de

Germany

Schleswig-Holstein, Lower Saxony and North Rhine-Westphalia

Wiggers GmbH - Reha / Rollstuhl Werkstatt
Gerhard Stalling Strasse 47 B
D-26135 Oldenburg
Tel: (+49) 0441 36 111 302
Fax: (+49) 0441 36 111 309
Email: wiggers.gmbh@ewetel.net
Internet: www.minicrosser.info; www.ot-wiggers.de

Holland

Richard van Seenus Nederland B.V.
Postbus 1425, 1300 BK Almere
Bolderweg 6, 1332 AT Almere
Tel: +31 (0) 36 53 20 450
Fax: +31 (0) 36 53 21 308
Email: info@vanseenus.nl
Internet: www.vanseenus.nl

Norway

Medema gruppen AS
Postbox 133
N-1483 Skytta
Tel: +47 6706 4900
Fax: +47 6706 4990
Email: firmapost@medema.no
Internet: www.medema.no

Spain

Assessorament I Mobilitat. S.L.
Marqués de Monistrol, Local 3 (esq. C./ Girona, 29-31)
08980 Sant Feliu de Llobregat - Barcelona.
Tel: +34 93 666 55 23
Fax: +34 93 666 88 85
Email: info@assessoramentimobilitat.com
Internet: http://www.assessoramentimobilitat.com

Sweden

Minicrosser AB
Bultgatan 28
S-442 40 Kungälv
Tel: (+46) 303 245 200
Fax: (+46) 303 245 228
Email: info@minicrosser.se
Internet: www.minicrosser.se

Switzerland

Degonda Rehab SA
Industriestrasse 12
CH-3661 Uetendorf
Tel: +41 033 345 83 00
Fax: +41 033 345 83 01
Email: info@turbo-twist.ch
Internet: www.turbo-twist.ch

UK

Mini Crosser Ltd
The Coach House
Westonbirt
Tetbury
Glos
GL8 8QS
Sales, tel: +(44) 0800 043 0029
Service & Warranty, tel: +(44) 0800 169 9588
Email: sales@mini-crosser.co.uk
Internet: www.mini-crosser.co.uk

USA / Canada

Pride Mobility Products Corp.
182 Susquehanna Ave.
Exeter, Pa. 18643
USA
Tel: +1 800 800 8586
Fax: +1 800 800 1636
Internet: www.pridemobility.com



medemagroup